

COVID-19 IgM/IgG Antibody Test
Diagnostic Sensitivity and Specificity Study Report

Drafted by: Jun Zhang

Approved by: Jerry Zheng

Final report date: May 21, 2020

Management of the study: Artron Laboratories Inc.
R& D Department
Quality control departmen

Table of Contents

Title Page

Table of Contents

Study Summary

1.0 Purpose

2.0 Reference and Compliance

3.0 Materials

4.0 Experiment Design

5.0 Evaluation Criteria

6.0 Result

7.0 Conclusion

8.0 Report

Study Summary

The purpose of this study was to obtain accurate information regarding the Diagnostic Sensitivity and Specificity of Artron COVID-19 IgM/IgG Antibody Test at three different evaluation sites from a total of 1162 samples, including 285 SARS-COV-2 positive cases confirmed by RT-PCR and 877 SARS-COV-2 negative samples.

The first evaluation was carried out at Affiliated Hospitals of Chongqing medical University, PRC. A total of 125 serum/plasma samples from COVID-19 infected patients were used: these included 6 asymptomatic infections, 8 patients with symptoms within 7 days, 49 patients with symptoms within 8-14 days, 62 patients with symptoms over 14 days. In addition to this, 123 non-COVID-19 infected sera/plasmas collected before November 2019 and stored in the third Affiliated Hospital, Chongqing medical University were also tested. Among all the chosen samples, Artron COVID-19 IgM/IgG Antibody identified 124 COVID-19 IgM&/or IgG positive samples including 108 IgM positive and 114 IgG positive from 125 COVID-19 infected patients samples.

The Second evaluation was conducted through Otogenetics Corporation in the US at 5 different clinic locations. A total of 780 samples were collected including 89 RT-PCR confirmed SARS-COV-2 positive samples and 691 SARS-COV-2 negative. Artron COVID-19 IgM/IgG Antibody identified a total of 83 COVID-19 IgM&/or IgG positive samples including 78 IgM positive and 80 IgG positive from 89 COVID-19 infected patients samples.

The third evaluation was conducted by BC CDC. A total 134 samples from hospitalized patients included 71 RT-PCR confirmed SARS-COV-2 positive samples and 63 SARS-COV-2 negative samples were collected. Among all the chosen samples, Artron COVID-19 IgM/IgG Antibody identified 68 COVID-19 IgM&/or IgG positive cases including 67 IgM positive and 65 IgG positive from 71 COVID-19 infected patients samples.

Combine all the results from the three clinic centers: A total of 1162 samples including 285 of RT-PCR confirmed SARS-COV-2 positive sera/plasma/whole blood samples and 877 of RT-PCR confirmed SARS-COV-2 negative sera/plasma or clinic true sera/plasma (collected before Nov. 2019) were used to evaluate Artron COVID-19 IgM/IgG Antibody Test amongst all three clinical evaluations. Out of all the 285 positive samples, Artron COVID-19 IgM/IgG Antibody Test identified 269 of COVID-19 IgM&/or IgG positive cases including 253 of IgM positive; 259 of IgG positive. The diagnostic sensitivity for IgM test was 88.77%; for IgG was

90.88%; the combined sensitivity was 94.39%. The diagnostic specificity for IgM was 98.40%; for IgG was 99.77%; the combined specificity was 98.18%. The overall agreement for IgM and IgG was 96.04% and 97.59%, respectively. The combined overall agreement was 98.18%. The PPV for IgM and IgG was 94.76% and 99.23%, respectively. The combined IgM & IgG PPV was 94.39%. The NPV for IgM and IgG was 96.42% and 97.11%, respectively. The combined IgM & IgG PPV was 99.42%.

1. Purpose

To validate the diagnostic sensitivity and specificity of Artron COVID-19 IgM/IgG Antibody Test.

2. Reference and Compliance

- FDA, CE, CFDA, CMDR guidance for In vitro diagnostic medical device
- The present study conformed to all applicable laws and regulations.

3. Materials

- Positive Samples: Clinical samples collected and stored in local clinical laboratories. COVID-19 positive specimens were confirmed by RT-PCR with local government authorized tests
- Negative Samples: non-COVID-19 sera/plasma were collected before Nov. 2019 or samples from patients who have no exposure to SARS-COV-2 and no febrile, no respiratory symptoms and confirmed SARS-COV-2 negative by authorized RT-PCR tests.
- Local government authorized RT-PCR reagents
- Artron COVID-19 IgM/IgG Antibody Test, Lot:20200408-CF.

4. Study Design:

4.1. The clinical performance was evaluated in SARS-COV-2 infected blood specimens and non- SARS-COV-2 infected blood specimens from subjects in the chosen hospital.

Total at least 100 SARS-COV-2 positive blood samples from RT-PCR confirmed SARS-COV-2 infected patients and 200 non- SARS-COV-2 blood samples from non-febrile and non-respiratory patients should be collected; all the samples should be tested with Artron COVID-19 IgM/IgG Antibody Test.

4.2. Examiner and clinical laboratories

Evaluation Center 1: Clinical Laboratory Department, The third Affiliated Hospital, Chongqing medical University, PRC.

Evaluation Center 2: Otogenetics Corporation (Atlanta, GA, USA), (Detail see the attachment clinic information)

Evaluation Center 3: BC CDC

4.3. Test Procedure:

- All tests were performed by the clinical technicians in the clinical laboratory according to the manufacturer's instructions using the confirmed samples.
- The experiment followed the principle of random double blind.

- Visual interpretations of the results of COVID-19 IgM/IgG Antibody Test were made independently by the clinical technicians.
- The testing center was responsible for summarizing the results.

5. Evaluation Criteria

| | C Line | M Line | G Line | Test Result Interpretation |
|---|-------------|--------|--------|--|
| 1 | Not present | Any | Any | Invalid Test. The specimen must be retested with another device. |
| 2 | + | - | - | Valid Test, Negative for antibodies for SARS-CoV-2. |
| 3 | + | + | - | Valid Test, IgM positive for antibodies for SARS-CoV-2. |
| 4 | + | + | + | Valid Test, IgM and IgG positive for antibodies for SARS-Cov-2. |
| 5 | + | - | + | Valid Test, IgG positive for antibodies for SARS-CoV-2. |

6. Results

6.1 Results from Evaluation Center 1: Clinical Laboratory Department, The third Affiliated Hospital, Chongqing medical University, PRC.

A total 125 serum/plasma samples from COVID-19 infected patients including 6 asymptomatic infections, 8 patients with symptoms within 7 days, 49 patients with symptoms within 8-14 days, 62 patients with symptoms over 14 days were used. In addition to this, a total of 123 non-COVID-19 infected sera/plasmas were collected before November 2019 and stored in the Clinical Laboratory Department, The third Affiliated Hospital, Chongqing medical University as well. Amongst all the chosen samples, Artron COVID-19 IgM/IgG Antibody identified a total 124 COVID-19 IgM &/or IgG positive cases including 108 IgM positive and 114 IgG positive cases from 125 COVID-19 infected patients samples. The diagnostic sensitivity for IgM was 86.40%, for IgG was 91.20%; the combined sensitivity was 94.40%; 1 equivocal IgM false positive case from a total of 123 non-COVID-19 sera was observed; the diagnostic specificity for IgM was 99.19% and for IgG was 100%; the combined specificity was 99.19%.

Table 1 Summary for the test results of confirmed COVID-19 samples from clinic center1

| Plasma/serum (RT-PCR Confirmed) | | | | | |
|---------------------------------|-----------|--------|---------|--------------|----------------------|
| Days from onset | Specimen# | IgM(+) | IgG(+) | Combined(+) | Combined Sensitivity |
| >14 days | 62 | 54 | 61 | 62 | 100.0% |
| 8-14 days | 49 | 45 | 45 | 47 | 95.9% |
| ≤7 days | 8 | 5 | 4 | 5 | 62.5% |
| Asymptomatic infection | 6 | 4 | 4 | 4 | 66.7% |
| Total | 125 | 108 | 114 | 118 | |
| Sensitivity | | 86.40% | 91.20% | 94.40% | |
| | | | | | |
| Days from onset | Specimen# | IgM(+) | IgG (+) | Combined (+) | |
| ≤7 days Sensitivity | 8 | 62.5% | 50.0% | 62.5% | |
| >7days Sensitivity | 111 | 89.2% | 95.5% | 98.2% | |

Table 2 Diagnostic sensitivity and specificity from clinic center 1

| | | RT-PCR/Clinic truth | | |
|---------------------------------------|----------|---------------------|----------|-------|
| | | Positive | Negative | Total |
| Artron COVID-19 IgM/IgG Antibody Test | Positive | 118 | 1 | 119 |
| | Negative | 7 | 122 | 129 |
| | Total | 125 | 123 | 248 |

Diagnostic sensitivity: $118 / (118 + 7) \times 100\% = 94.4\%$

Diagnostic specificity: $122 / (122 + 1) \times 100\% = 99.19\%$

Overall agreement: $(118 + 122) / 248 \times 100\% = 96.77\%$

6.2 Results from Evaluation Center 2: Otogenetics Corporation, (Atlanta, GA, USA)

A total of 780 samples were collected including 89 RT-PCR confirmed SARS-COV-2 positive samples and 691 SARS-COV-2 negative samples. Amongst all the chosen samples, Artron COVID-19 IgM/IgG Antibody Test identified 78 IgM positive and 80 IgG positive cases out of a total of 83 COVID-19 IgM &/or IgG positive from 89 COVID-19 infected patients samples. The diagnostic sensitivity for IgM was 87.64%, for IgG was 89.89%, and the combined sensitivity was 93.26%; 9 of the IgM false positive and 2 of the IgG false positive cases from a total of 691 non-COVID-19 sera were observed and the diagnostic specificity for IgM was 98.70% and for IgG was 99.71%; the combined specificity was 98.41%.

Table3 Summary for the test results of confirmed COVID-19 samples from clinic center2

| Plasma/serum/whole Blood (Mol Confirmed) | | | | | |
|--|-----------|---------------|---------------|---------------|----------------------|
| Days from onset | Specimen# | IgM(+) | IgG(+) | Combined(+) | Combined Sensitivity |
| >14 days | 69 | 65 | 68 | 69 | 100.00% |
| 8-14 days | 8 | 7 | 8 | 8 | 100.00% |
| ≤7 days | 12 | 6 | 4 | 6 | 50.00% |
| Total | 89 | 78 | 80 | 83 | 93.26% |
| Sensitivity | | 78/89(87.64%) | 80/89(89.89%) | 83/89(93.26%) | |
| Days from onset | Specimen# | IgM(+) | IgG (+) | Combined (+) | |
| ≤7 days Sensitivity | 12 | 50.00% | 33.33% | 50.00% | |
| >7days Sensitivity | 77 | 93.51% | 98.70% | 100.00% | |

Table 4 Diagnostic sensitivity and specificity from clinic center 2

| | | RT-PCR confirmed/clinic truth | | |
|---------------------------------------|----------|-------------------------------|----------|-------|
| | | Positive | Negative | Total |
| Artron COVID-19 IgM/IgG Antibody Test | Positive | 83 | 11 | 94 |
| | Negative | 6 | 680 | 686 |
| | Total | 89 | 691 | 780 |

Diagnostic sensitivity: $83/(83+6) \times 100\% = 93.26\%$

Diagnostic specificity: $680/(11+680) \times 100\% = 98.41\%$

Overall agreement: $(83+680)/780 \times 100\% = 97.82\%$

6.3 Results from Evaluation center 3: BC Centre for Disease control(CDC)

A total 134 samples from hospitalized patients including 71 RT-PCR confirmed SARS-COV-2 positive samples and 63 SARS-COV-2 negative samples were collected. Among all the chosen samples, Artron COVID-19 IgM/IgG Antibody identified 67 IgM positive and 65 IgG positive out of a total of 68 COVID-19 IgM&/or IgG positive from 71 COVID-19 infected patients samples; the diagnostic sensitivity for IgM was 94.37%, for IgG was 91.55% and the combined sensitivity was 95.77%; 4 of the IgM false positive and 0 of the IgG false positive from a total of 63 non-COVID-19 sera were observed and the diagnostic specificity for IgM was 93.65% and for IgG was 100%; the combined specificity was 93.65%.

Table 5 Summary for the test results of confirmed COVID-19 from clinic center 3

| POCT(Confirmed positive) | | | | | |
|--------------------------|-----------|--------|---------|--------------|----------------------|
| Days from onset | Specimen# | IgM(+) | IgG(+) | Combined(+) | Combined Sensitivity |
| >14 days | 32 | 32 | 32 | 32 | 32/32(100%) |
| 8-14 days | 28 | 27 | 25 | 28 | 28/28(100%) |
| ≤7 days | 11 | 8 | 8 | 8 | 8/11(87.5%) |
| Total | 71 | 67 | 65 | 68 | 6/6(100%) |
| Sensitivity | | 94.37% | 91.55% | 95.77% | |
| Days from onset | Specimen# | IgM(+) | IgG (+) | Combined (+) | |
| ≤7 days Sensitivity | 11 | 72.73% | 72.73% | 72.73% | |
| >7days Sensitivity | 60 | 98.33% | 95.00% | 100.00% | |

Table 6 Diagnostic sensitivity and specificity from clinic center 3

| | | RT-PCR confirmed/clinic truth | | |
|---------------------------------------|----------|-------------------------------|----------|-------|
| | | Positive | Negative | Total |
| Artron COVID-19 IgM/IgG Antibody Test | Positive | 68 | 4 | 72 |
| | Negative | 3 | 59 | 62 |
| | Total | 71 | 63 | 134 |

Diagnostic sensitivity: $68/(68+3) \times 100\% = 95.77\%$

Diagnostic specificity: $59/(59+4) \times 100\% = 93.65\%$

Overall agreement: $(68+59)/134 \times 100\% = 94.78\%$

6.4 Summary for all the test results from the three evaluation centres

A total of 1162 samples including 285 of RT-PCR confirmed SARS-COV-2 positive sera/plasma/whole blood samples and 877 of RT-PCR confirmed SARS-COV-2 negative sera/plasma or clinic true sera/plasma samples (collected before Nov. 2019) were used to evaluate Artron COVID-19 IgM/IgG Antibody Test from three evaluation centers. Among all of the 285 positive samples, Artron COVID-19 IgM/IgG Antibody Test identified 253 of IgM positive and 259 of IgG positive out 269 of COVID-19 IgM&/or IgG positive samples. The diagnostic sensitivity for IgM was 88.77%, for IgG was 90.88% and the combined sensitivity was 94.39%. The diagnostic specificity for IgM was 98.40%, for IgG was 99.77% and the combined specificity was 98.18%. The overall agreement for IgM and IgG was 96.04% and 97.59%, respectively. The combined overall agreement was

98.18%. The PPV for IgM and IgG was 94.76% and 99.23%, respectively. The combined IgM and IgG PPV was 94.39%. The NPV for IgM and IgG was 96.42% and 97.11%, respectively. The combined IgM and IgG PPV was 99.42%.

Table 7 Summary for all the test results of SARS-COV-2 patients from three evaluation centres:

| Summary for the Sensitivity | | | | | |
|-----------------------------|-------------|-----------------|-----------------|-----------------|-------------|
| Days from onset | Specimen# | IgM(+) | IgG (+) | Combined (+) | Sensitivity |
| >14 days | 163 | 151 | 161 | 163 | 100.0% |
| 8-14 days | 85 | 79 | 78 | 83 | 97.65% |
| <7 days | 31 | 19 | 16 | 19 | 61.30% |
| Asymptomatic infection | 6 | 4 | 4 | 4 | 66.67% |
| Total | 285 | 253 | 259 | 269 | 94.39% |
| | Sensitivity | 88.77% | 90.88% | 94.39% | |
| Days from onset | Specimen# | IgM(+) | IgG (+) | Combined (+) | |
| ≤7 days Sensitivity | 31 | 19/31(61.29%) | 16/31(51.61%) | 19/31(61.29%) | |
| >7days Sensitivity | 248 | 230/248(92.74%) | 239/248(96.37%) | 246/248(99.19%) | |

Table 8 Summary for all the test results from three evaluation centres

| Artron COVID-19 IgM/IgG Antibody Test | RT-PCR confirmed /clinic truth | | | | | | |
|---------------------------------------|--------------------------------|------------------|------------------|-----------------------|-----------------------|-------------------------|------------------|
| | Positive(N=285) | | | Negative(N=877) | | | |
| | IgM(+) | IgG(+) | Combined (+) | False positive IgM(+) | False positive IgG(+) | Combined False Positive | True negative |
| Clinic Centre 1 | 108/125 | 114/125 | 118/125 | 1/123 | 0/123 | 1/123 | 122/123 |
| Clinic Centre 2 | 78/89 | 80/89 | 83/89 | 9/691 | 2/691 | 11/691 | 680/691 |
| Clinic Centre 3 | 67/71 | 65/71 | 68/71 | 4/63 | 0/63 | 4/63 | 59/63 |
| Total | 253/285 (88.77%) | 259/285 (90.88%) | 269/285 (94.39%) | 14/877 (1.60%) | 2/877 (0.23%) | 16/877 (1.82%) | 861/877 (98.18%) |

Table 9 Summary for IgM diagnostic sensitivity and specificity

| | RT-PCR confirmed | | |
|--|------------------|----------|-------|
| | Positive | Negative | Total |
| | | | |

| | | | | |
|---|----------|-----|-----|------|
| Artron COVID-19 IgM/IgG Antibody Test-IgM Testing | Positive | 253 | 14 | 267 |
| | Negative | 32 | 863 | 895 |
| | Total | 285 | 877 | 1162 |

Diagnostic sensitivity for IgM of Artron COVID-19 IgM/IgG Antibody Test:
 $253/(253+32) \times 100\% = 88.77\%$

Diagnostic specificity for IgM of Artron COVID-19 IgM/IgG Antibody Test:
 $863/(863+14) \times 100\% = 98.40\%$

PPV: $253/(253+14) \times 100\% = 94.76\%$

NPV: $863/(863+32) \times 100\% = 96.42\%$

Overall agreement: $(253+863)/1162 \times 100\% = 96.04\%$

Table 9 Summary for IgG diagnostic sensitivity and specificity

| | | RT-PCR confirmed | | |
|---|----------|------------------|----------|-------|
| | | Positive | Negative | Total |
| Artron COVID-19 IgM/IgG Antibody Test-IgG Testing | Positive | 259 | 2 | 261 |
| | Negative | 26 | 875 | 901 |
| | Total | 285 | 877 | 1162 |

Diagnostic sensitivity for IgG of Artron COVID-19 IgM/IgG Antibody Test:
 $259/(259+26) \times 100\% = 90.88\%$

Diagnostic specificity for IgG of Artron COVID-19 IgM/IgG Antibody Test:
 $875/(875+2) \times 100\% = 99.77\%$

PPV: $259/(259+2) \times 100\% = 99.23\%$

NPV: $875/(875+26) \times 100\% = 97.11\%$

Overall agreement: $(259+875)/1141 \times 100\% = 97.59\%$

Table 10 Summary for combined IgM&IgG diagnostic sensitivity and specificity

| | | RT-PCR confirmed/Clinic Truth | | |
|--------|----------|-------------------------------|----------|-------|
| | | Positive | Negative | Total |
| Artron | Positive | 269 | 16 | 285 |

| | | | | |
|---|----------|-----|-----|------|
| COVID-19 IgM/IgG Antibody Test | Negative | 16 | 861 | 877 |
| | Total | 285 | 877 | 1162 |

Diagnostic sensitivity of Artron COVID-19 IgM/IgG Antibody Test:
 $269 / (269 + 16) \times 100\% = 94.39\%$

Diagnostic specificity of Artron COVID-19 IgM/IgG Antibody Test:
 $861 / (861 + 16) \times 100\% = 98.18\%$

PPV: $269 / (269 + 16) \times 100\% = 94.39\%$

NPV: $861 / (861 + 5) \times 100\% = 99.42\%$

Overall agreement: $(259 + 861) / 1141 = 98.18\%$

7. Conclusion

The clinical evaluation was carried out for the clinical performance of COVID-19 IgM/IgG Antibody Test. at three different evaluation sites from a total of 1162 samples, including 285 SARS-COV-2 positive cases confirmed by RT-PCR and 877 SARS-COV-2 negative samples.

The first evaluation was carried out at Affiliated Hospitals of Chongqing medical University, PRC. A total of 125 serum/plasma samples from COVID-19 infected patients were used: these included 6 asymptomatic infections, 8 patients with symptoms within 7 days, 49 patients with symptoms within 8-14 days, 62 patients with symptoms over 14 days. In addition to this, 123 non-COVID-19 infected sera/plasmas collected before November 2019 and stored in the third Affiliated Hospital, Chongqing medical University were also tested. Among all the chosen samples, Artron COVID-19 IgM/IgG Antibody identified 124 COVID-19 IgM&/or IgG positive samples including 108 IgM positive and 114 IgG positive from 125 COVID-19 infected patients samples. The diagnostic sensitivity for IgM was 86.40%, for IgG was 91.20%; the combined sensitivity was 94.40%; 1 equivocal IgM false positive from total 123 non-COVID-19 sera was observed, the diagnostic specificity for IgM was 99.19% and for IgG was 100%; the combined specificity was 99.19%.

The Second evaluation was conducted through Otogenetics Corporation in the US at 5 different clinic locations. A total of 780 samples were collected including 89 RT-PCR confirmed SARS-COV-2 positive samples and 691 SARS-COV-2 negative. Artron COVID-19 IgM/IgG Antibody identified a total of 83 COVID-19 IgM&/or IgG positive samples including 78 IgM positive and 80 IgG positive from 89 COVID-19 infected patients samples. The diagnostic sensitivity for IgM was 87.64%, for IgG was 89.89%; the combined sensitivity was 93.26%. From a total of 691 non-COVID-19 sera, 9 of IgM false

positives and 2 of IgG false positives were observed; the diagnostic specificity for IgM was 98.70% and for IgG was 99.71%; the combined specificity was 98.41%.

The third evaluation was conducted by BC CDC. A total 134 samples from hospitalized patients included 71 RT-PCR confirmed SARS-COV-2 positive samples and 63 SARS-COV-2 negative samples were collected. Among all the chosen samples, Artron COVID-19 IgM/IgG Antibody identified 68 COVID-19 IgM&/or IgG positive cases including 67 IgM positive and 65 IgG positive from 71 COVID-19 infected patients samples. The diagnostic sensitivity for IgM was 94.37%, for IgG was 91.55%; the combined sensitivity was 95.77%. From a total of 63 non-COVID-19 sera, 4 of IgM false positives and 0 of IgG false positive cases were observed; the diagnostic specificity for IgM was 93.65% and for IgG was 100%; the combined specificity was 93.65%.

Summary of the clinical evaluation results:

A total of 1162 samples including 285 of RT-PCR confirmed SARS-COV-2 positive sera/plasma/whole blood samples and 877 of RT-PCR confirmed SARS-COV-2 negative sera/plasma or clinic true sera/plasma (collected before Nov. 2019) were used to evaluate Artron COVID-19 IgM/IgG Antibody Test amongst all three clinical evaluations. Out of all the 285 positive samples, Artron COVID-19 IgM/IgG Antibody Test identified 269 of COVID-19 IgM&/or IgG positive cases including 253 of IgM positive; 259 of IgG positive. The diagnostic sensitivity for IgM test was 88.77%; for IgG was 90.88%; the combined sensitivity was 94.39%. The diagnostic specificity for IgM was 98.40%; for IgG was 99.77%; the combined specificity was 98.18%. The overall agreement for IgM and IgG was 96.04% and 97.59%, respectively. The combined overall agreement was 98.18%. The PPV for IgM and IgG was 94.76% and 99.23%, respectively. The combined IgM & IgG PPV was 94.39%. The NPV for IgM and IgG was 96.42% and 97.11%, respectively. The combined IgM & IgG PPV was 99.42%.

8. Report

8.1 Original raw data is archived at Quality Control Department

8.2 The original final report is archived in Quality Control Department.